




1 Check contents

1. Vscan with probe
2. Docking station
3. USB 2.0 cable
4. Storage microSD card (Already installed in Vscan)
5. Battery
6. AC/DC converter & Adapter plugs
7. Quick card
8. Ultrasound gel
9. Soft case
10. CD-ROM Vscan gateway software
11. CD-ROM user manual
12. microSD card with factory SW (In a sealed envelope)

2 Insert battery

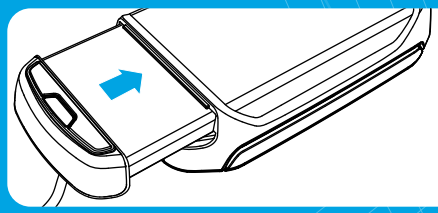

3 Charge battery

1. Select plug

UK, Hong Kong Singapore	Australia New Zealand	Korea
North America Japan	China	Continental Europe


2. Mount

3. Plug

Indicator lamp - Amber: battery charging
- Green: battery fully charged

Charge completely before first use.

4 Connect

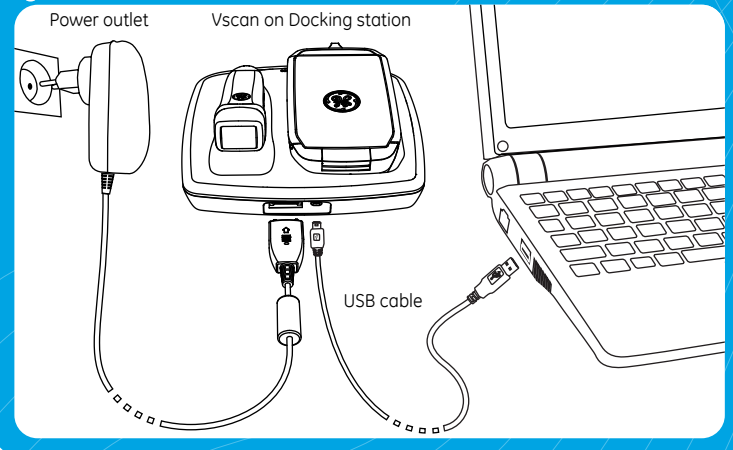

5 Activate Vscan & Vscan gateway software

The Vscan and Vscan gateway software must be activated before use. The Vscan activation will require the user to enter an activation key. The Vscan gateway software activation will require the user to enter a System ID.

Retrieve the activation key and the System ID.

Step 1



Log on to <https://vscan.gehealthcare.com> and select Register. Follow the on-screen steps to get activation key and System ID.

-or-



Call your local service contact. (Refer to user manual on CD for Hotline number.)

Activate Vscan.

Step 2

1. Select language.
2. Insert Activation key.
3. Adjust time.
4. Adjust date.
5. Ready.

Activate Vscan gateway software.

Step 3

1. Install Vscan gateway software.
2. Run Vscan gateway software and enter the System ID when requested.

The PIN code is required when retrieving the System ID in the Vscan portal.

Press if not already connected to the Vscan portal.

Enter the System ID.

Battery

- Fully charged.
- Partially discharged.
- Low: prepare to recharge.
- Discharged: recharge.
- Charging

MicroSD card

- Empty
- Partially full
- Nearly full
- Full: exchange or empty.
- No card

Menu

- Create examination.
- Make measurement.
- Select preset.
- Open Gallery.
- Configure.

Vscan CE 0470

United States law restricts this device to sale or use by or on the order of a physician.

Read and understand the user manual before using the Vscan.

Contraindication: The Vscan ultrasound unit is not intended for ophthalmic use or any use causing the acoustic beam to pass through the eye.



Vscan Quick Card

Scanning

Start Apply gel Scan Depth Freeze / Unfreeze Store

Start Color Color Gain Stop Color Return to b&w scanning

Presets Select Presets menu Select preset Voice notation Record Stop recording

Review & Store

Scan Freeze Select frame Store Still frame Play (Pause) Store Video

Measure

Press Menu Select Distance Place first point Place second point Store

Gallery

Press Menu Select Gallery Select exam Select file Review other files in exam